



FITTING GUIDE

BASE CURVE SELECTION*

CORNEAL CYLINDER	BASE CURVE
0.00 to 1.00	On Flat K
1.12 to 1.50	.25 steeper than Flat K
1.62 to 2.00	.50 steeper than Flat K
2.00 to 2.50	.75 steeper than Flat K
2.75 or more	discuss with consultant

*This nomogram is based on using the **recommended diameter**.
 Smaller Diameter — Fit .25D steeper
 Larger Diameter — Fit .25D flatter

DIAMETER SELECTION

BASE CURVE	DIAMETER		
	SMALL	RECOMMENDED	LARGE
7.10 mm to 7.38 mm	8.7	9.0	9.3
7.42 mm to 8.08 mm	9.0	9.3	9.6
8.13 mm to 8.49 mm	9.3	9.6	9.9

